

# Comfort

Love. Care.  
Comfort.



## Product Catalog

## 2023

Comfort – a specialistic rehabilitation stroller  
that brings relieve

# ABOUT COMPANY

We are a Polish company that for the last 35 years has been producing specialistic rehabilitation strollers Comfort. They are designed for children and adults suffering from cerebral palsy (CP), spiral muscular atrophy (SMA) or other afflictions causing palsies or limbs paresis

## ONE OF OUR BASIC PRINCIPLES IS TO MEET INDIVIDUAL EXPECTATIONS OF OUR CLIENTS.

While creating Comfort strollers we focus on their optimal influence on the patients. Bearing in mind how important the vestibular and proprioceptive systems are for the patient's growth, we pay particular attention to keeping the good body posture in a stroller.

## OUR STROLLERS ARE REGISTERED IN THE OFFICE FOR REGISTRATION OF MEDICINAL PRODUCTS, MEDICAL DEVICES AND BIOCIDAL PRODUCTS.

In 2005 we implemented the international system of standardisation ISO 1901:2000, which results in meeting the highest standards of production and customer service.

Our aim is to provide our clients with comfort in every sphere of their lives. That is why we actively support organizations caring for the disabled.

In compliance with the CSR (Corporate Social Responsibility) strategy introduced in our company, we get involved into numerous projects aiming at securing support and coming to help individuals with multiple and complex disabilities.





# FEATURES OF COMFORT STOLLERS



## Comfort

All reviews concerning Comfort strollers show that they are supremely comfortable for children, teenagers as well as adults. Comfort strollers are very easy to use.



## Reliability

We have 40 years of experience in producing specialistic strollers. Hence, we can offer a 25-year guarantee for the stroller frame.

Comfort strollers are reliability because we never use plastic elements.



## Support

Configurator of the Comfort stroller will help you to choose adequate equipment.

We can recomend solutions for the maximum subsidy. We can service your product both under/out of warranty within few days.



## Individuality

Indyvidual design of Comfort stroller may include shelf for respirator, profiled upholstery under the hump or special leg straps.



## Safety

The construction and balance of Comfort stroller guarantee stability and safety of its user.

Tube tyres and two-point brake used in Comfort stroller guarantee both a safe stop of the stroller.



## Practically

Optimal size enables you to fold it and place in a small car. Swivel wheels make it possible to move both in small spaces and in the field. Recument and semirecumbent option available.

Double stroller seat upholstery enables using it even when the outer layer needs to be cleaned. Construction of comfort strollers enable the child to sit at the table. You can choose from over a dozen of useful elements.

Comfort stroller can be adjusted to be used together with a stairclimber TGR Jolly Junior. Comfort stroller ensure the body stability due to the optimal seat profile and foam pads, which have a positive infulence on priproception.

Comfort stroller can be used together with stabilizing pillows Stabilo.



# SPECIAL COMFORT STROLLER FOR CHILDREN

Comfortable, safe, reliable, functional, fully ergonomic, a special stroller for children ensuring comfort and mobility every day. Perfect as a rest and stroller to move around the city, in the field and indoors



## DEEP ADJUSTABLE HEADREST

It stabilizes the correct arrangement of the neck and head with muscle laxity of the nape or with permanent degenerative changes.

## SPECIAL SEAT PROFILE

It guarantees stabilization and very high comfort even during many hours spent in a stroller. It will act positive for proprioception.

## STABILIZING PADS

Enables stabilization of the body, positively affecting proprioception.

## ABDUCTION WEDGE

Drains the lower limbs, gently correcting them knee alignment without undue pressure point.

## ADJUSTABLE FOOTREST

Easy adjustment of the footrest position horizontally and vertically depending on the needs user.

## SWIVEL FRONT WHEELS

They make it easier to drive and maneuver the stroller, especially in narrow and confined spaces and in the field.

## BACKREST WITH THE OPTION LYING

Smooth adjustment of the backrest angle to the position lying as an option.

## COMFORTABLE BELTS

Belts to stabilize the lower limbs, pelvis, chest depending on user needs.

## UPHOLSTERY

Made of easy-to-clean material.

## CONVENIENT DRIVEWAY TO THE TABLE

The construction of the stroller allows the child to sit down conveniently at the table (for selected models).

## PNEUMATIC WHEELS

Solid wheels used in the smallest trolleys guarantee full maintenance-free.



## Available colors





# SPECIAL COMFORT STROLLER FOR ADULTS

Comfortable, safe, reliable, functional, fully ergonomic, special stroller for adults providing comfort and mobility every day. Perfect as a rest and stroller to move around around the city, in the field and indoors



## DEEP ADJUSTABLE HEADREST

It stabilizes the correct arrangement of the neck and head with muscle laxity of the nape or with permanent degenerative changes.

## SPECIAL SEAT PROFILE

It guarantees stabilization and very high comfort even during many hours spent in a stroller. It will act positive for proprioception.

## STABILIZING PADS

Enables stabilization of the body, positively affecting proprioception.

## ABDUCTION WEDGE

Drains the lower limbs, gently correcting them knee alignment without undue pressure point.

## ADJUSTABLE FOOTREST

Easy adjustment of the footrest position horizontally and vertically depending on the needs user.

## SWIVEL FRONT WHEELS

They make it easier to drive and maneuver the stroller, especially in narrow and confined spaces and in the field.

## BACKREST WITH THE OPTION LYING

Smooth adjustment of the backrest angle to the position lying as an option.

## COMFORTABLE BELTS

Belts to stabilize the lower limbs, pelvis, chest depending on user needs.

## DOUBLE UPHOLSTERY

Enables using it even when the top layer needs to be cleaned. Used fabrics that can be cleaned easily.

## EVEN FOR VERY LARGE PATIENTS

The construction of the trolley allows for safe transport of patients weighing up to 130 kg.

## PNEUMATIC WHEELS

Solid wheels used in the smallest trolleys guarantee full maintenance-free.



## Available colors



**Comfort**

# PURPOSE




	Stroller for children				Stroller for adults			
	Baby [1]	Size [3]	Size [4]	Size [5]	Size [6]	Size [6+]	Size [7]	Size [8]
<b>The purpose of the stroller</b>								
Age of the patient	0 - 3	3 - 6	6 - 12	6 - 14	from 14	from 14	from 14	from 14
Maximum patient weight [kg]	30	35	50	50	75	75	90	130
Patient height [cm]	110	140	145	155	150 - 165	160 - 180	160 - 180	160 - 180
<b>Stroller parameters</b>								
Seat width [cm]	33,5	35	38	38	41	41	44,5	52,5
Seat length/depth [cm]	27	33	37	41	45	48	48	48
Backrest height [cm]	55	66,5	70	76,5	78	83	83	83
Maximum opening angle against the backrest [°] standard position	up to 120°	up to 135°	up to 130°	up to 135°	up to 120°	up to 120°	up to 120°	up to 120°
Footrest height adjustment range [cm]	23 - 26	27,5 - 33,5	31,5 - 42	32 - 42,5	36 - 46	36 - 46	36,5 - 46,5	36,5 - 46,5
Max footrest opening angle in relation to the seat [°]	110°-155°	110° -145°	100° -135°	95° -135°	100° - 140°	95° - 150°	90° -130°	90° -130°
Approximate weight of trolley [kg]	15	21	23	23	26,2	26,5	28	32
Stroller dimensions after folding (length x width x height) [cm]	97,5 x 55 x 50,5	89,5 x 62 x 55	94 x 64,5 x 51,5	98 x 64,5 x 55	112,5 x 69,5 x 56	122 x 69,5 x 55	122 x 70 x 54	122 x 70 x 54
Stroller dimensions (length x width x height) [cm]	111 x 55 x 92	123 x 62 x 96	121,5 x 64,5 x 94,5	122,5 x 64,5 x 94,5	141 x 69,5 x 95	143 x 69,5 x 95	148 x 70 x 95	148 x 70 x 95





# EQUIPMENT

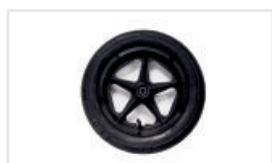


**Fixed wheels 12,5" x 2,25**  
(Cat. no.: ST)



**Seatbelts**  
(Cat. no.: ACPASBEZPR)



**Double upholstery**  
(Cat. no.: ACTAPW)

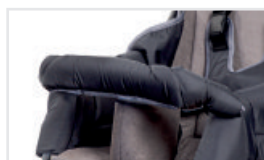

**Barrier**  
(Cat. no.: ACBAR)



**Inter-body wedge (abductive)**  
(Cat. no.: ACKLIN)



**Bag for the stoller**  
(Cat. no.: ACTORPWW)



**Headrest**  
(Cat. no.: ACZAG)

# FUNCTIONAL SOLUTIONS



**Full swivel wheels**  
(Cat. no.: SK1)



**Adjustable handle**  
(Cat. no.: RŁ)



**Backrest adapted to the hump**  
(Cat. no.: ACTAPG)



**Anti-tilt protection**  
(Cat. no.: ACUPP)



**Backrest tilted to a lying or reclining position [°]**  
(Cat. no.: OL)



**Raised armrests**  
(Cat. no.: PP)

# ADDITIONAL ACCESSORIES



**A booth with an apron**  
(Cat. no.: BUDA)



**Thigh belts**  
(Cat. no.: ACPASU)



**Cape**  
(Cat. no.: ACPEL)



**Side cushions**  
(Cat. no.: ACPODUB)



**Sleeping bag**  
(Cat. no.: ACSPI)



**Lumbar supports - pelottes**  
(Cat. no.: ACPODPL)



**Umbrella**  
(Cat. no.: PARASOL.KOL.)



**Lumbar pillow**  
(Cat. no.: ACPODUL)



**Table**  
(Cat. no.: ACSTOL)



**Seat cushion**  
(Cat. no.: ACPODUS)



**Handle bag**  
(Cat. no.: TORBA.KOL.)



**Outer upholstery**  
(Cat. no.: ACTAPW)



**Hand protectors**  
(Cat. no.: ACOCHRAC)



**Footrest protector**  
(Cat. no.: ACOCHPOD)



**Calf belts**  
(Cat. no.: ACPASŁ)



**Ankle belts**  
(Cat. no.: ACPASK)



**Foot belts**  
(Cat. no.: ACPASS)



# ADDITIONAL INFORMATION

We invite you to use the electronic media library and to follow our social media.



## YouTube

Channel presenting Comfort film materials.

[youtube.com/channel/UCj8kE1tWS87ry4be080OpHQ](https://youtube.com/channel/UCj8kE1tWS87ry4be080OpHQ)



## Facebook

Brand profile informing about new products, current activities of the company and charity activities.

[facebook.com/Comfort-w%C3%B3zki-rehabilitacyjny-105814814917073](https://facebook.com/Comfort-w%C3%B3zki-rehabilitacyjny-105814814917073)



## Instagram


An image profile showing the use of prams, the use of accessories and functional solutions.

[instagram.com/comfortpolska](https://instagram.com/comfortpolska)




# MEASUREMENT TABLE

How to correctly measure the User? In order to choose the right one the size of the trolley, the measurement of the User should be made.

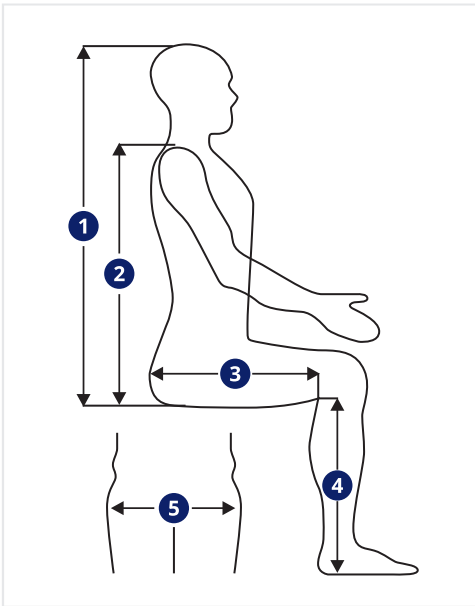

The most important are 5 dimensions:

- ❶ The length of the body from the seat to the top of the head
- ❷ The length of the body from the seat to the shoulders
- ❸ The length of the body from the back to the knee well
- ❹ Shin length (from the knee to the sole)
- ❺ Hips (distance between the widest places)

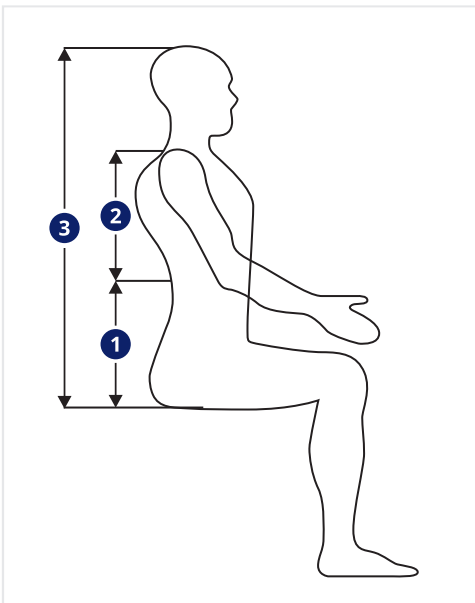

If there is a hump, it is necessary accurate dimensioning.

- ❶ Length of the lumbar spine
- ❷ The length of the hump from the shoulders to the end of its occurrence
- ❸ The length of the body from the seat to the top of the head

In addition, it should be indicated whether the hump is left- or right-sided.





# USER TESTIMONIALS

“

I visited personally on-live comfort facility in Poznan Poland to see and buy one stroller for my specially abled child, he is suffering from cerebral Palsy and not able to walk by himself, based on his needs I have been given plenty of options for my child needs, i selected comfort (size 5) for my kid. I met Laura she supported me through the process, i am very happy to get us equiped for my kid. I strongly recommend comfort stroller.

*Anshul Ansh's parent*

“

Our Son – Hubert has always used a wheelchair. He is currently 30 years old. Only once we had a stroller other than Comfort. We are very satisfied with our choice. The stroller from your company is very comfortable, we have no reservations. Our son loved his stroller so much, he got used to it as if they formed one duo – we do not want to change the stroller for another. Comfort is perfect and we completely confirm this.

*Hanna Hubert's mom*

“

Since kindergarten, when Alexander was a toddler, we already had a pram from your company – Comfort. For some time we changed prams for Olek to choose the most suitable one. Over the years of experience, we are happy to return to the pram from Comfort. It is the best choice, we are 100% convinced. We are returning after 20 years to have the Comfort pushchair again forever.

*Aleksandra*

“

The Comfort pram is one of the best we have had so far. We have a lot of confidence in you and have a good opinion. If a stroller then only from Comfort!

*Margaret*

“

I am a big advocate of Comfort wheelchairs. As long as the person being cared for is in control of the sitting position it is the ideal option. For those who lack this stability once the wheelchair has been suitably adapted, it too can fulfil its function and be a huge support to both the person using it and the carer. I always use Comfort strollers – especially as a 'respite' / pram.

*Dariusz*

“

Our Comfort stroller is the perfect choice! It protects exemplarily in spontaneous movements, I am reassured about the safety of my child.

*Maciej*

“

The 'Comfort' wheelchairs have been used by students at our school for years. They are great for older people whose deformities do not allow them to be positioned in other wheelchairs. A plus point is the anti-tipping mechanism, which prevents people with severe athetosis from tipping the trolley over.

*Monika*

“

Working with Comfort has been good. We have products that have served us for many years and our favourites. For this reason, we are ordering more products.

*Nursing home*

“

We really liked the Comfort pushchair, it fulfils its role, has a nice colour and lots of accessories to choose from. For me personally, it's a relief pushchair. I am calm about my daughter and she travels comfortably.

*Ania's mum*

“

The Comfort stroller is very good, nice and provides us with the opportunity to get out of the house. We appreciate your ideas, the stroller is the whole world to us.

*Marianna*

“

My son Olaf is an adult. I purchased two Comfort strollers so that I can better manage daily needs. I take my son out of the flat myself, then I had to carry the pushchair and now it's easier for me. I move the boy, the pushchair is waiting, we leave with joy. The pushchair meets our expectations, one model for home and another for trips out.

*Olaf's dad, Michael*

“

We are very pleased with your company Comfort. The stroller is very good durable. The material of the pram is very good non-staining. Very comfortable as well as very well made. Comfort is an intelligent and understanding company. The wheelchair is made to meet the requirements of the customer, which meets the needs of a disabled person. I recommend this company wholeheartedly.

*Joanna*

Love. Care. Comfort.

Comfort

**WSR COMFORT sp. z o. o.**  
ul. Głazowa 43, 60-116 Poznań, Poland



[www.comfort-reha.pl/en](http://www.comfort-reha.pl/en)



+48 531 539 355



Our catalogue

